




Lympha Press USA



LYMPHA PANTS II™

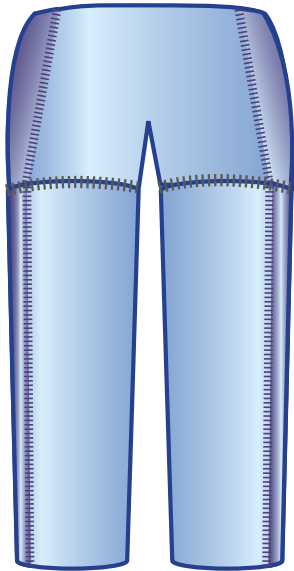

UNIQUE modular design
LEG SLEEVES zip on to torso garment
TREATS one leg or two
EASY to wear – just like a pair of pants!



Lymphedema therapy with torso and abdominal treatment designed on the principles of manual lymph drainage

Leg lymphedema can be associated with lymphedema in the torso, groin, or abdomen. Until now, manual lymph drainage was the only option for torso therapy.

Lympha Pants II™ - Torso treatment based on the principles of manual lymph drainage. Designed in conjunction with leading manual lymph drainage therapists, **Lympha Pants II™** treat the entire lower body, including the abdomen.

Lympha Pants II™ can be used to treat one leg or two simultaneously, with complete therapy of the entire abdomen and groin.




Use on one leg or two, together with complete abdominal therapy

Features:

- Complete therapy for legs and abdomen
- 12-24 overlapping chambers, for directional flow
- Easy to put on and take off
- Easy to clean, for good health
- Non-stretch fabric ensures correct compression every time
- Treats both legs simultaneously

LYMPHA PANTS II™

Lympha Pants II™ are fully adjustable, for patients big and small

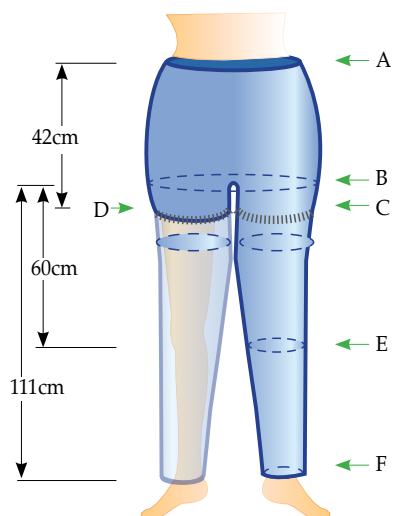

Sizing Tables

Garment Measuring Point	Bilateral pants				
	No Expanders	1 expander one side only	2 expanders one per side	4 expanders two per side	Increase per additional expander
Waist (A)	130.0	148.0	166.0	202.0	18.0
Hips at crotch (B)	127.0	142.0	157.0	187.0	15.0
Upper Thigh (C&D)	73.5	87.5	87.5	101.5	14.0
E (1/3 from end)	63.0	75.0	75.0	87.0	12.0
F (end)	52.5	61.5	61.5	70.5	9.0

Garment Measuring Point	Unilateral pants				
	No Expanders	1 expander one side only	2 expanders one per side	4 expanders two per side	Increase per additional expander
Waist (A)	124.5	142.5	160.5	196.5	18.0
Hips at crotch (B)	121.5	136.5	151.5	181.5	15.0
Upper Thigh, treated leg (C)	73.5	87.5	87.5	101.5	14.0
Upper Thigh, untreated leg (D)	68.0	68.0*	82.0**	96.0**	14.0
E (1/3 from end)	63.0	75.0	75.0	87.0	12.0
F (end)	52.5	61.5	61.5	70.5	9.0

*assumes single Leg (long) expander is used on treated side

** assumes GA (short) expander is used on untreated side

Lympha Pants II™ fit wide variations in size, with internal fasteners and zip-in expanders to adjust circumference.



Lympha Press Optimal®

Calibrated gradient compression with Pretherapy™ based on the principles of manual lymph drainage.



Lympha Press® 20I Max

Calibrated gradient compression, with adjustable pressure in three zones.

Contraindications

Use of the Lympha Pants II™ is not recommended in the presence of one or more of the following conditions: Known or suspected deep vein thrombosis or pulmonary embolism; during the inflammatory phlebitis process; acute infection of the affected limb; decompensated cardiac failure; severe arteriosclerosis or other ischemic vascular disease; any circumstance where increased venous and lymphatic return is undesirable. Due to movement of fluids in the body when using the device, be careful with patients who have heart disease. Use of high pressure is not recommended for patients with peripheral occlusion disease. The abdominal area should not be treated during pregnancy. If Lympha Pants II™ is to be used during pregnancy, the air cells over the abdomen must be disconnected prior to use.

Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.

Lympha Press USA Ltd. 232 Park Avenue, Manalapan, NJ 07726

Tel: 1-888-596-7421, 732-792-9677, Fax: 732-792-9745

info@lymphapress.com www.lymphapress.com