



Lympha Press USA

COMFYSLEEVE™

Features:

- Complete upper body treatment
- Targeted axilla therapy
- Easy to wear
- Easy to clean
- Fully adjustable compression



Simplifying treatment for postmastectomy lymphedema

Lymphedema therapy doesn't have to be a hassle. It can be a quiet, restful part of your daily routine. **COMFYSLEEVE™** by Lympha Press® is designed for effective lymphedema therapy in the clinic or in the comfort of home.

COMFYSLEEVE™ is easy to put on and take off, so you can complete your daily therapy quickly and efficiently. No fussy pieces to assemble for each session - our arm and torso garments zip together and are worn just like a jacket. It's also easy to keep clean. And our Lympha Press® pumps are fully adjustable - your medical professional can prescribe a treatment program that is exactly right for you.

Use COMFYSLEEVE™ with the following systems:



Lympha Press Optimal™

- Calibrated sequential gradient compression with Pretherapy™ based on the principles of manual lymph drainage. Medicare coded E0652.



Lympha Press® 20 Max

Calibrated sequential gradient compression with adjustable pressure in three zones, for comfortable and safe therapy. Medicare coded E0652.

Indications:

- Secondary lymphedema
- Postmastectomy lymphedema
- Primary lymphedema
- Venous insufficiency
- Dysfunction of the “muscle pump,” wound healing, traumatic edema

Contraindications:


- Known or suspected DVT or pulmonary embolism
- Acute infection
- Decompensated cardiac failure
- Severe arteriosclerosis or other ischemic vascular disease
- Any circumstance in which increased venous and lymphatic return is undesirable
- The abdominal area should not be treated during pregnancy

Easy to clean, important for good skin care.
Easy to use, for maximum compliance.

COMFYSLEEVE™ treats the entire arm, shoulder, and one side of the back, chest and abdomen. Its sequential compressive massage action activates the lymphatics to absorb fluids and proteins as it coaxes fluids around blockages, into healthy areas for uptake.

COMFYSLEEVE™ is designed to meet the needs of patients with upper extremity lymphedema.

- Unique design makes it easy to wear and remove.
- Treats arm, shoulder, back, chest and abdomen.
- Targeted therapy for axilla drainage.
- Calibrated gradient compression - fully adjustable to meet special needs.
- Easy to clean, for good hygiene.

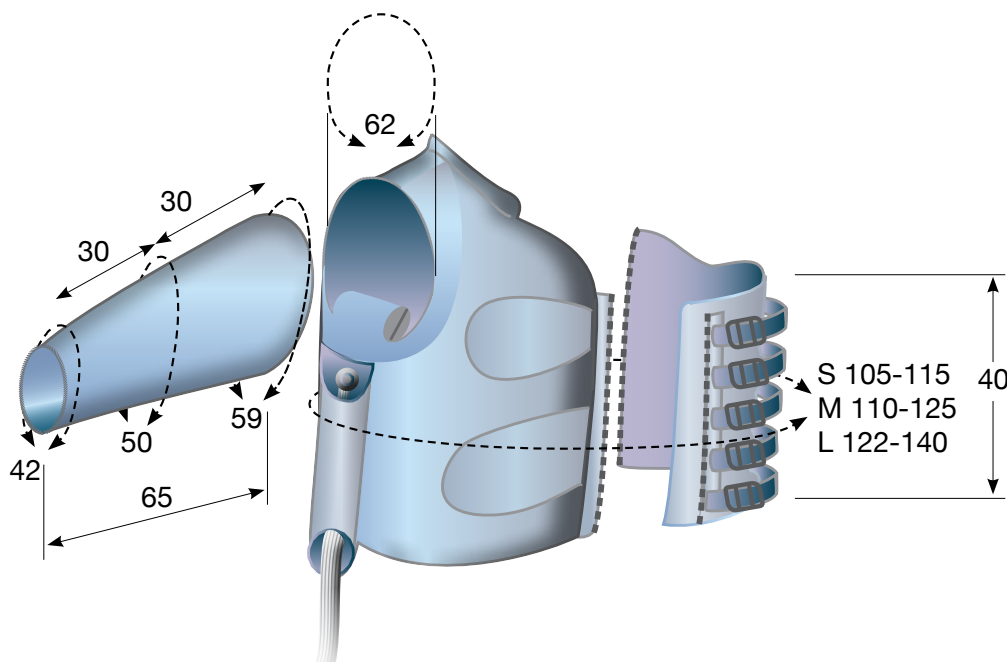

Garment (Specify LEFT or RIGHT)	Catalog No.	Torso Dimensions	
		Min.	Max.
ComfySleeve I-75 Small	Left L41LSI75L Right L41LSI75R	105	115
ComfySleeve I-75 Medium	Left L41LM175L Right L41LM175R	110	125
ComfySleeve I-75 Large	Left L41LL175L Right L41LL175R	122	140

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

Distributed by: Lympha Press USA Ltd. 232 Park Avenue, Manalapan, NJ 07726
Tel: 1-888-596-7421, 732-792-9677, Fax: 732-792-9745
info@lymphapress.com www.lymphapress.com