



Lympha Press USA

# the Petite Basic System™



HCPCS Code E0651



- Four fully overlapping compression cells
- Rapid cycle time
- Adjustable pause
- Lock feature to prevent tampering



## The Petite Basic System™

Manufactured by the makers of Lympha Press®, **The Petite Basic System™** is a sequential pneumatic compression system designed for treatment of venous disorders and dysfunction of the “muscle pump.”

**The Petite Basic System™** has *four fully overlapping compression cells* to ensure distal to proximal flow and comfortable compression. Pressure and pause times are easy to adjust. The *Lock feature* prevents accidental tampering, for patient safety.

The **Petite Basic System™** garments have a zipper closure for easy wear and are available in a wide range of sizes, with expanders for a temporary increase in circumference.

**The Petite Basic System™** is designed to give quality treatment at home. Its ease of use encourages patient compliance, while its unique overlapping chamber design provides comfortable and effective treatment.



# the Petite Basic System™



## Garments

### Measurements in cm




| Cat. No. | Garment Size & Length | Garment Dimensions |    |    |    |    |    |    |               | With Expander |    |    |    |    |
|----------|-----------------------|--------------------|----|----|----|----|----|----|---------------|---------------|----|----|----|----|
|          |                       | A                  | B  | C  | D  | E  | F  | L  | Expander Size | A             | B  | C  | D  | E  |
| 2250     | Leg S 50 cm           | 57                 | 54 | 46 | 43 | 44 | 26 | 50 | A             | 67            | 63 | 50 | 48 | 48 |
| 2275     | Leg S+ 73 cm          | 66                 | 56 | 46 | 43 | 44 | 26 | 73 | BT            | 76            | 66 | 54 | 48 | 48 |
| 2285     | Leg M 84 cm           | 70                 | 60 | 47 | 43 | 44 | 26 | 84 | CT            | 80            | 70 | 55 | 48 | 48 |
| 2295     | Leg M+ 95 cm          | 80                 | 72 | 60 | 48 | 37 | 32 | 95 | ET            | 90            | 82 | 72 | 53 | 41 |
| 2365     | Leg L 65 cm           | 81                 | 73 | 60 | 56 | 54 | 37 | 65 | BT            | 91            | 83 | 70 | 59 | 58 |
| 2375     | LegL+ 75 cm           | 85                 | 76 | 61 | 56 | 54 | 37 | 75 | CT            | 95            | 86 | 69 | 59 | 58 |
| 2385     | Leg L++ 85 cm         | 90                 | 80 | 63 | 56 | 54 | 37 | 85 | DT            | 100           | 90 | 71 | 59 | 58 |

**A:** Sleeve upper circumference, **L=** Sleeve length. **B - F** are additional measurements.

**Always use the Petite Basic System™ according to your medical professional's instructions.**

## Features:

- Fully variable pressure range from 20 to 80 mmHg
- Lock button to prevent tampering
- Adjustable pause time
- Bilateral application when needed
- Four fully overlapping compression cells
- Color coding to eliminate error
- Quiet, safe operation
- Lightweight and portable

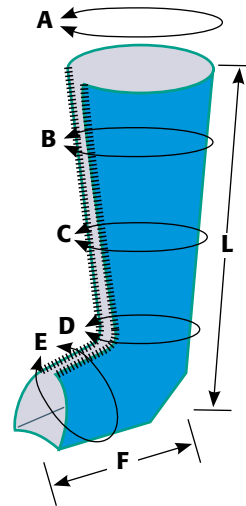

## Pump Specifications

|                       |                     |
|-----------------------|---------------------|
| <b>CYCLE</b>          | Sequential          |
| <b>PRESSURE RANGE</b> | 20-80 mmHg          |
| <b>SIZE</b>           | 10.2x5.1x3.9 inches |
| <b>WEIGHT</b>         | 5.1 lb.             |
| <b>ELECTRICAL</b>     | 115V, ~50/60Hz.     |

## Use of the Petite Basic System™ is not recommended in the presence of:

Severe arteriosclerosis or other ischemic vascular disease, unbalanced cardiac insufficiency, extreme deformity of the limbs, known or suspected DVT or thrombophlebitis, gangrene, dermatitis, untreated or infected wounds, severe inflammation of the affected limb, recent skin grafts, or in the presence of any pain or numbness.

**Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.**

**Distributed by: Lympha Press USA Ltd. 232 Park Avenue, Manalapan, NJ 07726**

**Tel: 1-888-596-7421, 732-792-9677, Fax: 732-792-9745**

**info@lymphapress.com www.lymphapress.com**